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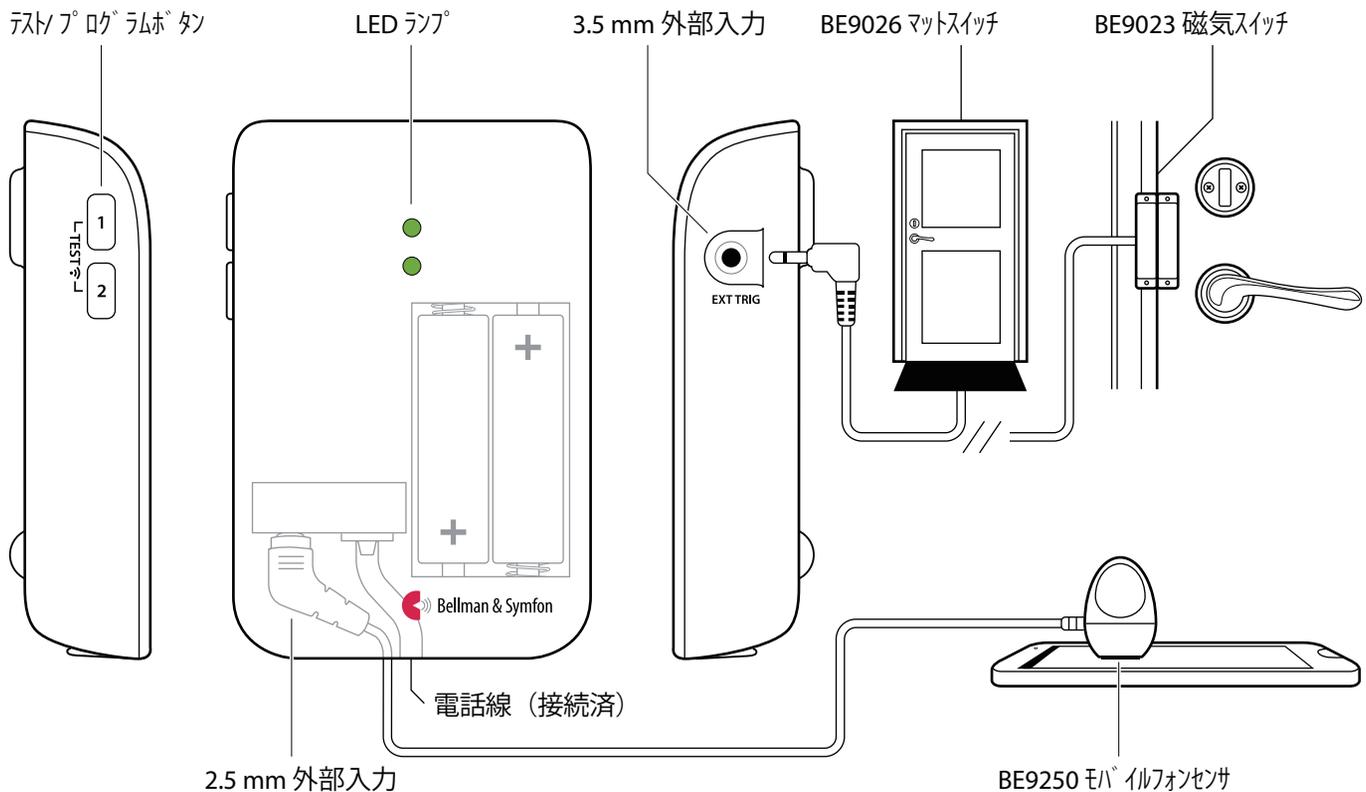
Transmitter

Receiver signal pattern

Source	LED	LED	Sound	Vibration	Flash
■ Landline phone	Green, top	Yellow light	1×ring signal, low	Medium	■□■□ Yes
■ Mobile phone sensor	Green, top	Yellow blinks	2×ring signal, high	Medium	■□■□ Yes
■ Other accessory	Green, bottom	Green light	1×door chime, low	Slow	■□□□ Yes

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Transmitter

Receiver signal pattern

Switch	Source	LED	Sound	Vibration	Flash
	Landline phone / test button Mobile phone sensor* Other accessory	Yellow light Yellow blinks Green light	1×ring signal, low 2×ring signal, high 1×door chime, low	Medium Medium Slow	■□■□ Yes ■□■□ Yes ■□□□ Yes
	Landline phone / test button Mobile phone sensor* Other accessory	Yellow light Yellow blinks 2×green blinks	1×ring signal, low 2×ring signal, high 2×door chime, low	Medium Medium Slow	■□■□ Yes ■□■□ Yes ■□□□ Yes
	Landline phone / test button Mobile phone sensor* Other accessory	Yellow light Yellow blinks 3×yellow blinks	1×ring signal, low 2×ring signal, high 1×ring signal, high	Medium Medium Medium	■□■□ Yes ■□■□ Yes ■□■□ Yes
	Landline phone / test button Mobile phone sensor* Other accessory	Yellow light Yellow blinks 2×orange blinks	1×ring signal, low 2×ring signal, high Baby melody	Medium Medium Fast	■□■□ Yes ■□■□ Yes ■■■■■■ Yes
	Landline phone / test button Mobile phone sensor* Other accessory	2×yellow blinks Yellow light 3×orange blinks	2×ring signal, low 1×ring signal, low Baby melody	Medium Medium Fast	■□■□ Yes ■□■□ Yes ■■■■■■ Yes
	Landline phone / test button Mobile phone sensor* Other accessory	2×yellow blinks Orange blinks Green blinks	2×ring signal, low Baby melody 2×door chime, high	Medium Fast Slow	■□■□ Yes ■■■■■■ Yes ■□□□ Yes
	Landline phone / test button Mobile phone sensor* Other accessory	Orange blinks 3×yellow blinks 2×green blinks	Baby melody 1×ring signal, high 2×door chime, low	Fast Medium Slow	■■■■■■ Yes ■□■□ Yes ■□□□ Yes
	Landline phone / test button Mobile phone sensor* Other accessory	3×yellow blinks 2×yellow blinks Green blinks	1×ring signal, high 2×ring signal, low 2×door chime, high	Medium Medium Slow	■□■□ Yes ■□■□ Yes ■□□□ Yes

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